



SYSTEM OF NARCOTICS IDENTIFICATION

INSTRUCTIONS FOR USE


TEST 0

FOR: GHB (Gamma Hydroxybutyrate)

Test Function

1. This reagent system presumptively identifies GHB.

How to Use Test 0

1. If sampling a powdered substance, remove the clip and insert into the test pack an amount of powdered suspect material that would fit inside this circle. 
2. Reseal with the clip and tap the pouch gently to assure the material falls to the bottom of the pack.
2. If sampling a liquid substance, place 2 small drops of liquid in the pouch using a disposable pipette (dropper).
3. With the printed side of the test pack facing you, break the ampoule by squeezing the center of the ampoule with the tips of the thumb and forefinger. Agitate gently and observe color or color changes.

Interpretation of Resulting Colors

1. A rapidly developing green from amber (natural color) indicates the presence of GHB.
2. Other colors, or no color at all, may indicate the presence of other drugs. Start with Test A and refer to the NIK® Identidrug™ Chart.

CAUTION

- A. This test contains alcohol. Do not use near an exposed flame, spark or other ignition source.
- B. Do not ingest.
- C. **Antidote:** In case of contact, immediately flush eyes or skin with water for at least 15 minutes while removing contaminated clothing and shoes. If swallowed, do not give emetics. If conscious give large quantities of water to dilute the alcohol. Call a physician.



13386 International Parkway
Jacksonville, Florida 32218
904.485.1836 FAX: 904.741.5407
© 2105 Safariland, LLC

A Brand of the Safariland Group